

# KIM 8 SLIMMING SYSTEM

CAVITATION+VACUUM+ MULTIPOLE RF+LASER

Manual



Thanks for using our company's products, in order for the proper and prolong use of our products, we sincerely suggest to comply with the followings:

Please read the instructions in detail and keep it well for further reading and consulting.

Please follow the instructions to fit and operate the machine properly.

Please do not remove and change any accessories of the machine.

Strongly forbids unauthorized personnel to open and remove the mainframe of the device.

The device's maintenance and repair should be operated by trained professionals.

Please contact us if there is any queries during operation.

## Working principle

### 1. Wave Fat System

With our collective strong sound wave head, strong sound waves of 40KHZ is emitted to vibrate fat cells at top speed and produce numerous vacuum air pockets inside and outside the fat cells. Robustly impacting the fat cells to generate introverted blast and disintegrate triglyceride into glycerol and free fatty acids. Then RF waves at frequency of 1MHZ is used for exhausting the integrated glycerol and free fatty acids through hepatoenteral circulation. Finally, vacuum RF and energy electrode are used for positioning and tightening fat. In physics, this is known as "cavitation". Micropore introverted blast inside and outside cell would lead to enhanced molecular motion and a higher energy level. This will finally cause fat cells rupture and thereby achieve the effects of body building and losing weight.

### 2. What are the benefits of Lipo Laser?

It adopts the latest innovation in non-surgical, non-invasive laser lipolysis. Lipo laser emits low levels of laser energy which creates a chemical signal in the fat cells breaking down the stored triglycerides into free fatty acids, glycerol and water molecules. This is similar to the natural response initiated by the body when it needs to use its stored energy reserves. These free fatty acids is then transported throughout the body by the lymphatic system to supply the body with energy. Again just as the body would respond in times of caloric scarcity.


A period of exercise in post treatment would ensure the complete metabolism of these free fatty acids. Shrinking the fat cells and delivering the body shape result our patients are looking for. The laser treatment is a non-destructive methodology aimed at targeted fat reduction which means excess fat deposits can be treated anywhere on the body and the fat cells are not destroyed as a result of the treatment. Targeted fat reduction and circumferential reduction has many advantages over other body shaping technologies. Here are just a few!

## Advantages

1. Safe, minimally-invasive procedure - it uses low levels of visible red laser light to create a safe and painless stimulation similar to the natural chemical response in our body to free up reserved energy resources stored in fat. It is safe, effective and shows immediate results.
2. Easy to operate, treatments are passive and require only the correct positioning of treatment pads. It will not compromise office staff time or productivity and won't effect life styles and work after treatment. It has a fast healing time.
3. Treat all skin types and parts of body such as neck, breasts, back, knees, hips, thighs, bottoms and arms
4. Accelerate tissue coagulation to avoid muscle slack after slimming.
5. No bleeding, no pain, no side-effect.
6. Economical and fast return on your investment.

## SKETCH of the CONTROL PANEL

Select the appropriate treatment and install the corresponding handle to proceed to the operation interface

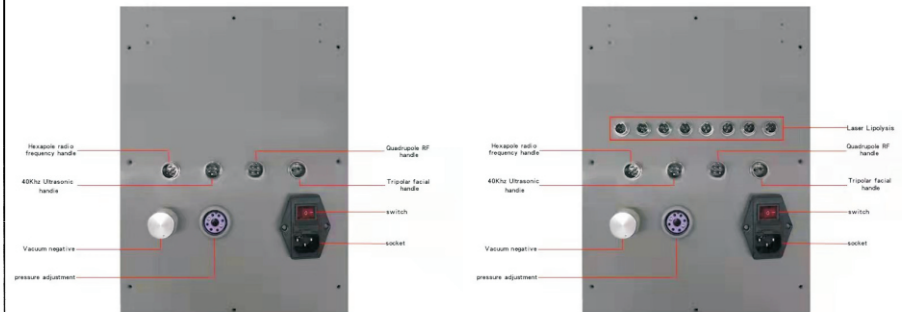

Package Including:  
 Tripolar facial handle \*1  
 Quadrupole RF handle \*1  
 Hexapole radio frequency handle \*1  
 40Khz Ultrasonic handle \*1  
 Vacuum Bipolar RF handle \*1  
 Bracket \*3  
 Laser Lipolysis \*8  
 Power cord \*1  
 belt \*2


## . Operation schematic drawing of Tripolar RF work head for face.

1. Massage the mandible center in circles to produce deep heat.

2. Massage lower jaw in lines.

3. Massage the triangle zone of both sides of the face in circles.

4. Massage from jaw to angulus oris to ear in lines.

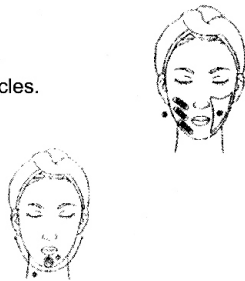

## UNSUITABILITY

1. Pregnant women or women during menses.
2. Epileptic
3. Patients with malignancy.
4. Patient whose wound after operation has not healed up.
5. Acute inflammation or epidemical patients.
6. Those with heart diseases or with heart pacemaker.
7. Those with kidney (gall-stone) disease.
8. Those with embedded metal object or silica gel.
9. Those in menses, under birth control, with emiction incontinence period, or undergone belly operation.
10. Those with genetic hypersensitivity.

## NOTICE

1. Be sure to use the special ultrasonic gel.
2. Handle the heads with care, avoid knocks.
3. Use and prepare enough gel to avoid overheating the heads.
4. Don't linger too long on a single region and avoid the bones.
5. Don't use disinfectants to disinfect the heads, use wet cotton or dry towel instead.
6. Make sure the power supply is connected and plugged well.
7. If the machine is not for use for a long period of time, switch it off and unplug the power supply.
8. Operators and patients should not wear any metallic jewellery or items.
9. The machine should be rested every hour of continuous operation for about 10 minutes.
10. During the operation, be sure not to accept other treatments.

## 1.40Khz Ultrasonic handle (Cavitation)

Strong sound wave explosion fat head - with collective strong sound wave head, strong sound wave of 40KHZ may be emitted to human body for impacting fat cells fiercely and causing friction motion between fat cells. This may cause effective consumption of calories and moisture in fat cells and reduce the size of fat cell. What's more, sound wave vibration may cause fierce impact of fat cells to make them be exploded instantaneously, reduce the amount of fat cells and thereby achieve the effects of removing fat.

There is 4 mode of A, B, C and D as described as follows:

"A" is continuous working for 3 sec and discontinuous for 3 sec.


"B" is continuous working for 2 sec and discontinuous for 2 sec.

"C" is continuous working for 1 sec and discontinuous for 1 sec.

"D" is continuous working for 0.5 sec and discontinuous for 0.5 sec.

Operate with the 40KHZ cavitation head for 15-20 minutes with mainly the spiral movement on body

Select the mode to operate using the "-" "+" selection.

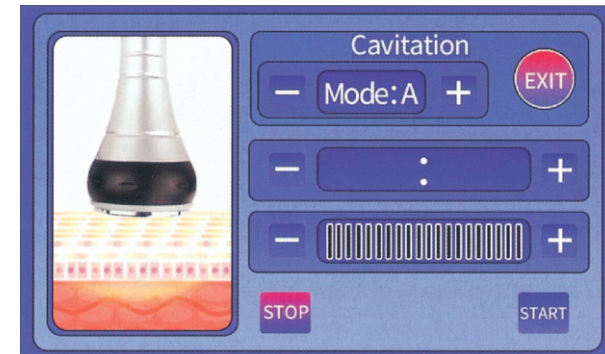

Press "-" "+" to adjust the duration of operation.

Below the duration selection is the intensity selection.

Clicking "-" to decrease the strength or click "+" to increase it.

Press "START" to start the strong sound wave explosion fat head.

Method : Use the probe to massage the specific part of skin to be treated circularly after applying the slimming cream for about 30 to 40 minutes. It could quicken the blood circulation in the treated part of skin and has the effect of decomposing fat.

Matters needing attention:


1. Apply slimming cream or cooling gel to the part of skin to be treated before the treatment.
2. Keep close contact between the probe and the skin.
3. Adjust the intensity from low level to high level.
4. Please do not irradiate eyeball directly with colored light.
5. Operate with the tripolar RF head for 15-20 minutes with mainly the spiral movement on face.



## Instrument Functions and Treatment Principles

### 2. Vacuum Bipolar RF handle (Vacuum & Bipolar RF)

Dissolve fat, enhance lymphatic drainage, firm skin and improve skin elasticity. Body-shaping equipment that adopts the most advanced high-speed turbine negative pressure technology and innovative three-dimensional conversion technology combining with microcomputer chip controlled patent roller axis. It has an anti-geocentric attraction effect on the skin. That is attracting and outspreading the fibrillar connective tissue of different cortexes through the rhythmic massage and effectively break down subcutaneous fat and reduce cellular tissue accumulation of fat. Coupled with the use of the patents methodology of massaging deeply with roller over different parts of the body to bring up fat layer in dermis curium and subcutaneous. It releases micro-vascular lymphatic vessels at the same time promoting metabolism and converts fat cells in a passive movement to fatty acids. Toxins within the cells are also eliminated from lymphatic system. In vitro the detoxification enhances the function as many as four times. The skin will become more elastic and luster. The perfect "healthy body sculpture management" expert.

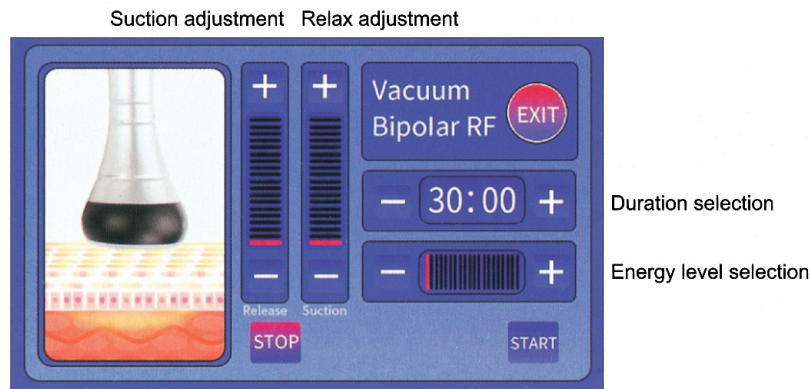

Panel : Press "RF CHANNEL" in the display screen to select vacuum bipolar RF output, select the suitable intensity (adjust according to the customer's acceptable level, 3-6 is recommended) and press "START" key to begin operation.

Method: Use the probe to massage the specific part of skin circularly after applying the slimming cream for about 30 to 40 minutes. This would quicken the blood circulation in the part of skin to be treated and has the effect of decomposing fat.

#### Note:


1. Apply slimming cream or cooling gel to the part of skin to be treated before the treatment.
2. Keep close contact between the probe and the skin.
3. Adjust the intensity from low level to high level.
4. Please do not radiate eyeball directly with colored light.
5. Operate with the bipolar vacuum head for 15-20 minutes with mainly spiral movement on body.

## Users Guide

1. Operation schematic drawing of 40K cavitation explode work head  
Massage the fat of thigh in circles



2. Operation schematic drawing of sextupole RF laser head for body  
Waist and abdomen care (25 minutes, 2-3 days/time)

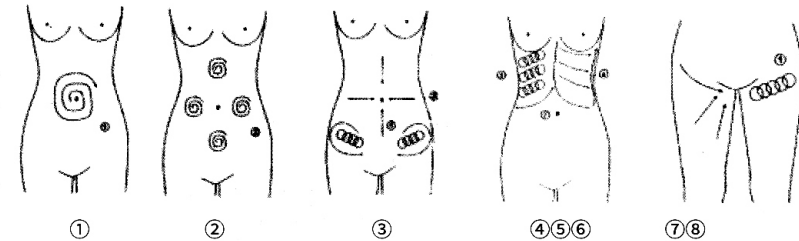


- ①. Massage to navel as center and gradually expanding the range
  - ②. Massage chest nest, both sides of former abdomen and below the navel (in diamond formation) from inside to outside in circles.
  - ③. Concentrated on the fat from 4 directions the navel as center
  - ④. Massage from inner rib to outer edge in circles.
  - ⑤. Massage from end of pelvis to groin in circles.
  - ⑥. Massage back and forth from inner rib to outer edge in straight line.
  - ⑦. Lift the Hip and the lower part of the body care (30minutes, 2-3 days/time) use the 50mm probe
  - ⑧. Concentrated from hip to groin in circles or straight line. Repeat massaging from inner to outer edge. Recycle the motion.
3. Operation schematic drawing of Tripolar RF work head for face.
1. Massage the mandible center in circles to produce deep heat.
  2. Massage lower jaw in lines.
  3. Massage the triangle zone of both sides of the face in circles.
  4. Massage from jaw to angulus oris to ear in lines.
  5. Massage from jaw to angulus oris to ear in lines.



## 6.Laser Lipolysis

Diode Lipo Laser emits low levels of laser energy which creates a chemical signal in the fat cell, breaking down the stored triglycerides into free fatty acids and glycerol and releasing them through channels in the cell membranes. The fatty acids and glycerol are then transported around the body to the tissues that will use them during metabolism to create energy. This process of fatty acid release is a natural response of the body when the body needs to use stored energy reserves, thus it will not create any unnatural reaction in the body nor does it cause any damage to surrounding structures such as skin, blood vessels and peripheral nerves. A period of exercise post treatment will ensure the complete metabolism and thus eliminate from the body the freed fatty acids. This 5mw lipo laser system can adopt light to penetrate deep level of skin to stimulate metabolism for effective slimming treatment. Tied laser board band at the site needing treatment and use laser for 20 minutes.

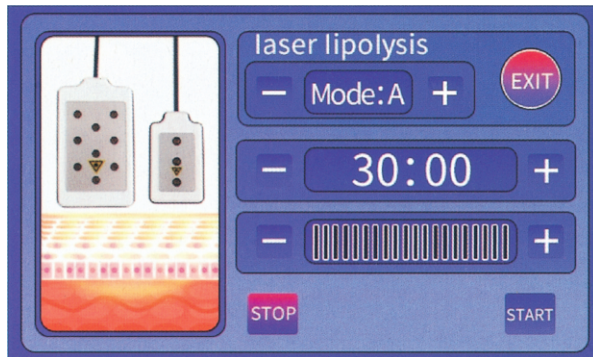


### Operation Precaution

1. Beautician should wear insulated gloves during operation.
2. Do not wear any jewellery or decorative accessories both for the beautician and the users.
3. The electrode probe must be fully in contact with skin.
4. Do not stagnant the RF electrode, keep it in continuous motion slowly.
5. Skin around eyes and forehead is very sensitive, the RF output intensity should be kept relatively low in such areas.

## 3.Tripolar facial handle (Tripolar RF)

Enhance rejuvenation by the use of positive and negative micro-current that activate energy that release skin cells so that rapid synthesis of fibroblasts collagen and skin elasticity. Muscle tension is stimulated and increased, gradually eliminate wrinkles and prevent skin aging, restoring skin elasticity and facial improvement.

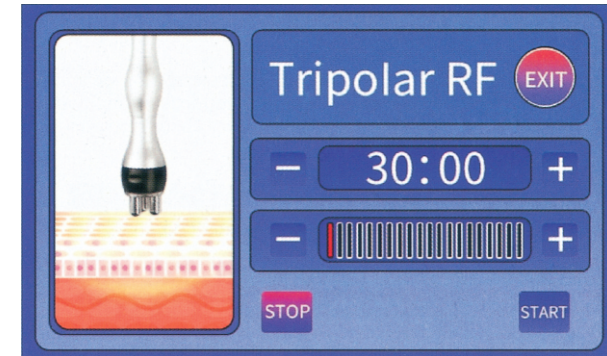


Press "-" "+" to adjust the time, clicking "+" to increase the RF intensity "-" to decrease RF intensity. Select the suitable intensity (adjust according to the customer's acceptable level, 3-6 is recommended) and press on "WORK" key to start. Spread gel or wrinkle smoothing essence on the treated positions (generally 20-30G each use), operate with the facial RF head for 15-20 minutes with mainly spiral movement on face and around the eyes.

#### 4.Sextupole body RF handle (RF Body )

Being integrated with the most advanced RF technology, the instrument can directly reach the deep-seated fat layer with the incisive positioning ability of RF. In the fast-active state, fat cell tissue may generate friction heat, increase local temperature and remove excess fat and toxin from the body through sweat gland, enterohepatic circulation and lymph. Ultimately achieve the effect of dissolving fat. With controlled depth technology, inlaid diamond particles of different sizes may rub skin along its pattern, adjust suction strength by means of vacuum draw and contact with strong negative pressure force to directly explode thick fat.

The effects are very obvious: Enhance blood circulation, accelerate the decomposition of fat, take away wastes, enhance the effect of removing cellulite, effectively improve the state of areolar tissue, remove stubborn fat and improve the sufficient absorption condition of diet products.

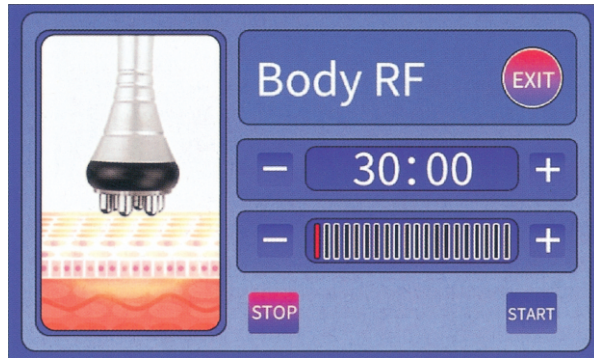

Press "-" "+" to adjust the duration and clicking "+" increase the RF intensity or "-" to decrease RF intensity. Press "START" to begin the operation of negative pressure vacuum RF head. Apply gel to the negative pressure vacuum RF head, aim at the area for losing weight and operate in pulling technique (drag fat areas at the position for losing weight to the position where it is closer to the lymph gland) for about 20 minutes.

#### Notes:

1. Spread gel or essence on the treated area evenly.
2. Start with low-energy. We suggest to test for the proper energy level to use on the back of clients before using on the to be treated areas.
3. Do not use essential oil as the medium.
4. In operation, start pulling only after negative pressure head is attached to skin.
5. Turn off the instrument probe when it is not in use.

#### 5.Quadrupole RF handle - (RF Face)

A capacitive coupling electrode is used for transmitting radio-wave energy and generating an electric field for entering into subcutaneous tissue through the skin surface. In the high-frequency

of 450khz, this electric field changes its polarity for 450,00 times per second. To response to the rapid change in electrode, directions of electrical particles the skin are also changed. By this time, natural electric resistance in the subcutaneous tissue moves and generates heat energy. As the papillary dermis collagen may immediately shrink when the temperature is within the range from 60 to 70 Celsius degree. After wrinkle treatment, client may immediately sense the skin tightening effects as it is being lifted and firmed.

When collagen is produced continuously, thickness and density of the skin papillary dermis may be increased to remove wrinkles, eliminate scars, restore skin elasticity and gloss and make it be blonde and smooth.

While collagen is increased, fresh skin is generated in the position of treatment and wrinkles are removed in large amount. In addition, when cortex without elasticity or that with thick horny layer in the area with wrinkles is separated, surrounding skin will also be renewed.



Press "-" "+" to adjust the duration. Click "+" to increase the RF intensity or "-" to decrease RF intensity, select the suitable intensity (adjust according to the customer's acceptable level, 3-6 is recommended) and lastly press "START" key to start.

Method: Use the probe to massage the specific part of skin to be treated circularly after applying the slimming cream for about 30 to 40 minutes. It could quicken the blood circulation in the treated part of skin and has the effect of decomposing fat.

#### Matters needing attention:

1. Apply slimming cream or cooling gel to the part of skin to be treated before the treatment.
2. Keep close contact between the probe and the skin.
3. Adjust the intensity from low level to high level.
4. Please do not irradiate eyeball directly with colored light.
5. Operate with the tripolar RF head for 15-20 minutes with mainly the spiral movement on face.